

# Yes! RAPID RESULT PREGNANCY TEST



**EARLY DETECTION**  
RESULTS 7-10 DAYS AFTER CONCEPTION

## Please read this leaflet before you begin the test

- **EARLY DETECTION:** Can be used as early as 7-10 days after conception. (7 days before next period due in a 28 day cycle).
- **SIMPLE TO USE:** It is best if you test your urine first thing in the morning. This is when the levels of the pregnancy hormone show higher.


1. Remove test from foil wrapper and take off replaceable cap.
2. Hold the tester in your urine stream for a full **15 seconds**. Do not allow urine to get on the test or control windows. You can collect your urine in a clean, dry container if you prefer, then place the tester tip in urine for the full **15 seconds**.
3. Once you have done this, remove from the urine, replace the cap and lay it down on a flat surface.
4. Wait **3 minutes** for the red lines to appear. If no red line appears wait another minute. (Test must be read within 10 minutes).
5. **Read the result:**

- A red line must appear in the control (C) window. This shows the test has worked.
- If there are two red lines - one (C) and one (T) then you are **PREGNANT**.
- If only one red line shows at the (C) then you are **NOT pregnant**.


*The red line at (T) may be very faint but this still shows a pregnancy result.*



### PREGNANT

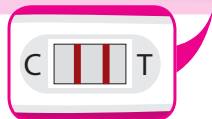

### NOT PREGNANT

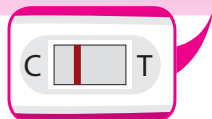

## How to interpret the result and next steps

**PREGNANT** - A pregnant result means your urine contains hCG, a hormone your body makes when you are pregnant. If your test says you are pregnant you should see your doctor who can advise you on what steps you should take next.

**NOT PREGNANT** - You may not be pregnant, or the level of pregnancy hormone may not yet be high enough to be detected, or you may have miscalculated the day your period was due. If you tested early, test again when your period is due. If your period is overdue, test again after 3 days. If you still get a **not pregnant** result but your period is still late then see your doctor because there could be health-related reasons as to why you haven't had your period.

**99% ACCURATE. DETECTS PREGNANCY QUICKLY AND EASILY.**

Simple and easy to use. A one-step answer in minutes.

## How does the test work?

The **Yes! Rapid Response Pregnancy Test** detects hCG, the hormone produced during pregnancy. The amount of this pregnancy hormone increases as your pregnancy progresses. This test is an early detection test designed to detect 10mIU/ml of the hCG and should be able to tell you if you are pregnant 7-10 days after conception (about 7 days before your next period is due if you have a 28-day cycle). The level of hormone is greater first thing in the morning so it is best to test your first urine of the day.

## How accurate is the test?

This test is over 99% accurate if used the day before your next period is due. However, it is an early detection test and can pick up levels of the hormone (hCG) 7-10 days after conception. Hormone levels in women do vary and some women may not have detectable levels of the hormone on the first day they test for it. This is why **at least 2 tests** are included in the pack so if necessary, a second test can be used.

## What do I do if the result says I am pregnant?

A positive pregnancy test result shows that levels of hCG have been detected. You should see your doctor to confirm, as this test is so sensitive some women may test positive even if they are not pregnant (this is usually due to a hormone called pituitary hCG or a medical condition such as an ovarian cyst or ectopic pregnancy). It is also important for you to receive medical advice for proper care and counselling.

## What do I do if the test says I am not pregnant?

This would mean you may not be pregnant or it is too soon because your levels of hCG are too low. You may also have miscalculated your menstrual cycle, or when you thought you may have conceived. If you still haven't had your period, or still think you are pregnant, then retest with a second test after three days. If you retest, and the result is still negative, then you should contact your doctor as there may be a health related reason for why your period is not happening.

**Further product information sheets are available at [www.smithbiomed.com](http://www.smithbiomed.com)**

TAPS No: PP9265

## What happens if the control line doesn't come up?

The result should appear within three minutes. If not wait another minute. If the lines have not appeared after 10 minutes or the control line is not present, then the test is invalid. This may be due to insufficient urine on the tester. You should retest with a new test, carefully following the instructions. You may want to consider collecting the urine in a clean dry container and immerse the tester in the urine for 15 seconds.

## What if the test line is very faint?

Even if the test line is faint it means a positive pregnancy result. The closer you test in relation to when your next period is due the stronger the line should become.

## Will the result change if I leave it for some time?

The test needs to be read after 3 minutes but within 10 minutes.

## Do drugs or medication affect the test?

This test should not be affected by hormone therapies containing clomiphene, by alcohol, painkillers, antibiotics or the contraceptive pill. It will only be affected by drugs containing the pregnancy hormone hCG.

## I have tried getting pregnant but have not yet succeeded. Is there anything I can do?

You are most likely to get pregnant during the 2-3 days around when you are ovulating. To be able to tell when this is happening use the **Yes! Baby Now Ovulation Test** or the **Yes! Fertility Test**. This kit contains all you need to have the best chance of conceiving- ovulation tests, pregnancy tests, a sperm concentration test and an ovulation chart and special ovulation thermometer.

Who can I contact if I have more questions?

For further information or advice on using the **Yes! Rapid Response Pregnancy Test**, either contact the pharmacy you purchased the tests from **OR** contact **Smith Biomed (NZ) Ltd** on 0508 BIOMED (246 633). Address all emails to [info@smithbiomed.com](mailto:info@smithbiomed.com).